



Soft Cervical Collar Instructions

THIS PRODUCT INTENDED FOR THE APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. PROPER FIT AND APPLICATION OF THIS PRODUCT MUST BE PERFORMED BY A MEDICAL PROFESSIONAL. THESE DIRECTIONS ARE GUIDELINES ONLY AND ARE NOT OFFERED AS MEDICAL RECOMMENDATIONS.

Applying cervical collars:

Center the support under your chin. Attach Velcro in the rear. **DO NOT** over tighten.

To Remove: Unfasten the Velcro closures and reapply Velcro neatly to collar.

Washing Instructions:

Hand wash support in mild soapy cool water. Rinse thoroughly and drip dry on a flat surface. **Do Not** use solvents or chlorine bleach. **Do Not** put in the dryer. **Do Not** iron.

Persons with broken skin or known susceptibility to dermatitis should avoid contact with this material. If a rash or skin irritation develops, discontinue use immediately and consult a physician. Support **Should Not** be worn while sleeping. Support **Should Not** be worn by persons with phlebitis or circulatory problems. **Never** wear support over an open wound.

Notice

While every effort has been made to obtain maximum compatibility of function, strength, durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. U.S. Orthotics disclaims any warranty, expressed or implied, that these products prevent injuries.

